


STEP 1 INSERTION




- Patient legs should be flexed
- Device is folded along the short axis by squeezing the wings between the thumb and fingers
- Tube attachment must point upwards during insertion (if the tube is facing downwards, it is likely to kink)

STEP 2 PLACEMENT

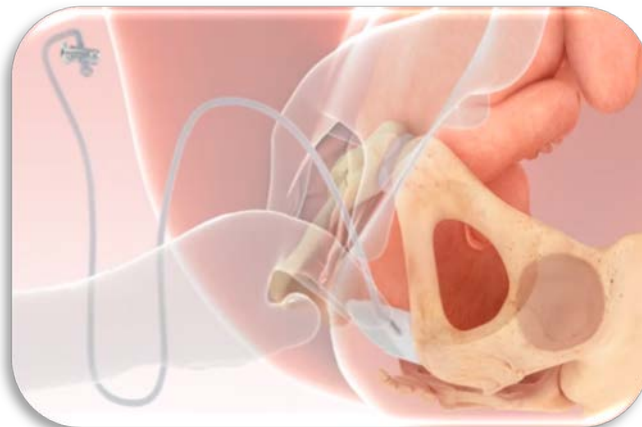

- Insert vaginally, and push the device posteriorly, towards the coccyx
- Placement is similar to a posterior ventouse cup
- The balloon surface of the device should be touching the fetal head

STEP 3 LEGS FLAT

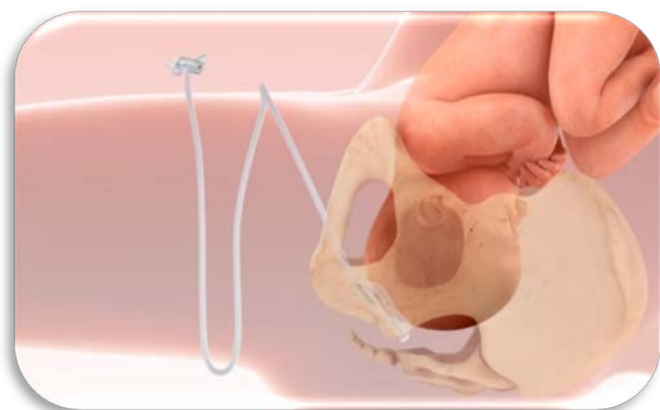

- Lay the legs flat on the operating table (otherwise it can be expelled or displaced if legs are open)

STEP 4 INFLATE



- Inflate with 180ml of saline using the 60ml syringe provided (three full syringes)
- Maximum volume should not exceed 300mls

Contact details for placing an order, or any queries, info@fetalpillow.com or www.fetalpillow.com

This quick reference guide does not include all of the information necessary for selection and use. Please see instructions for use for complete product and handling details.