


1. Insertion




- Fold the baseplate along the short axis ensuring balloon is on the inside
- Tube attachment must be at superior end (pointing upwards)
- Insert into the vagina

2. Placement

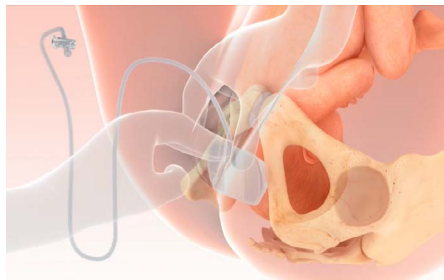

- Ensure balloon surface is in contact with the fetal head
- Push device posteriorly, towards the coccyx
- Placement is similar to a posterior Ventouse/vacuum cup

3. Legs Flat

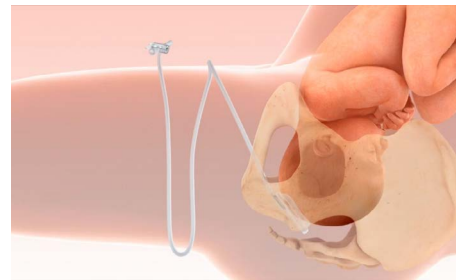

- Place patients legs flat on the table before inflation

4. Inflation



- Use the syringe to inflate using 180ml of sterile saline
- Close tap after filling to prevent fluid from leaking

5. Deliver baby

- Perform Cesarean section using standard technique

6. Deflation and removal

- Once baby has been delivered open the tap to drain the fluid
- Remove the device at the end of the procedure

Resources

For further guidance on how to use Fetal Pillow have a look at the following

Complete our online course

[sign up](#)

Click here for steps for use animation

[watch now](#)



Download the Fetal Pillow app

Scan this QR code using your mobile phone to access